ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:
The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS: This DonJoy Knee Brace with FourPoint® hinge is designed to support the knee and provide protected range of motion following ACL, PCL, ACL/PCL deficiencies, ACL reconstruction, collateral ligament, collateral ligament deficiencies with PCL involvement and MCL and/or LCL ligament. Soft-goods/semi-rigid designed to restrict motion through the knee joint. Provides medial protection of a limb or body segment for acute and prophylactic care.

CONTRAINDICATIONS: Do not use if you are allergic to any of the materials contained in this product.

WARNINGS AND PRECAUTIONS:
• Loss of circulation, patient discomfort and patient re-injury are potential effects caused by device failure.
• ROM settings should not be changed without supervision of a medical professional.
• This product must be prescribed and fitted by a healthcare professional.
• The frequency, duration of use and directions for use should be determined by your healthcare professional.
• If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
• The support should be snug but not impair circulation.
• Do not use over open wounds.
• Do not use the device if it is damaged and/or the packaging has been opened.
NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATOR INFORMATION:
1. While sitting on the edge of a chair, step through the brace, flexing the knee to 45° position the brace as shown in (figure 1), centering the hinge about 1" (2.54 cm) above the top of the knee and slightly past midline of the leg. Make sure the brace is not twisted on the leg.
B. Secure the bottom calf strap first (#1), then the top thigh strap (#2).
2. Keeping the knee bent at 45 degrees, push both condyles (hinges) toward the back of the leg, as shown in figure 2.
D. Maintaining the position of the hinges: for ACL version secure the upper calf strap (#3) and the upper calf strap (#3) then upper calf strap (#4) on the back of the leg.
E. For ACL version forward slightly before fastening the front tibial (shin) strap (#4). For PCL version draw the hinges forward slightly before fastening the front thigh strap (#6). Do not pull the hinges beyond midline of the leg.
F. For ACL version next, secure the upper calf strap (#5). Be careful not to pull this strap too tight and risk circulation problems.
G. When strapping is complete, the hinge should center just above the top of the kneecap and slightly behind midline. To ensure that no slippage occurs during activity, it is recommended to do a few knee bends or walk for a few minutes, then re-tighten the straps following steps B-F or B-E for CI version.

INSTRUCTIONS FOR HINGE ADJUSTMENT:
• Your brace comes with 10° extension stops installed. To change the installed stops, first completely remove the stop (figure 7) (or Armor by DonJoy & Legend (figure 5), for 4TTITUDE, 4TTITUDE Advanced & Renegade (figure 6). Then remove the stops. Insert the new stops, align with the holes in the hinge plate, then reinsert the screw. Using a screwdriver, tighten the screws. Ensure that the screws are seated in the recess and completely tightened. Both medial and lateral stops must be identical in degree.
• Lateral slide shield hinge cover (standard on LEGEND). In order to remove the extension stop, it is necessary to flip the hinge and slide the protective shield cover off before removing the stop (figure 7).
• Extension Stops: 0°, 10°, 20°, 30° and 40°.
• Flexion Stops: 0°, 45°, 60°, 75°, and 90° (no flexion stops are installed upon shipment).

USE AND CARE:
• If during activity, re-tightening is necessary, re-tighten all straps following steps in the application instructions.
• All straps and pads may be trimmed to fit individual variations in limb circumference.
• The brace may be used in fresh or salt water. Following use in water, activities, drain thoroughly and rinse with clean, fresh water and then air dry.
• Hand wash your liners and condyle pads in cold water with mild detergent. AIR DRY ONLY. DO NOT HEAT DRY LINERS OR PADS.
• The hinge mechanism requires little maintenance in normal use. However, you may apply a dry lubricant such as a Teflon™ spray to the normal mechanism of the hinge. Dry lubrication is recommended following use in water.
• Periodically inspect hinge screws for tightness and retighten as needed.

CLEANING INSTRUCTIONS: Hand wash in water (86°F/30°C) using mild soap. Air dry.

MATERIAL CONTENTS: Frame- Aluminum, Hinge- Aluminum & steel, Strap tabs- Aluminum & nylon, Soft goods- Nylon & EVA.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

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INTENDED FOR SINGLE PATIENT USE.
 NOT MADE WITH NATURAL RUBBER LATEX.

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NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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