ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS: The PSA-2 consists of an anatomically shaped pillow with an arm sling, an adjustable, diagonally attached shoulder strap and a waist belt. All parts are attached to the cushion.

INDICATIONS

- Rotator cuff repairs
- Sub-acromial bursitis/impingement
- Acromioplasty
- Shoulder Arthroplasty
- Mobilisation after frozen shoulder

CONTRAINDICATIONS: None

WARNINGS AND PRECAUTIONS:

- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use this device if it is damaged and/or the packaging has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS:

The first application on the patient should always be made by a health care professional.

The health care professional will advise whether the patient may take the cushion off, e.g. for personal hygiene purposes.

1. Loosen both straps from the clip closures in order shorten them to the required length. For this purpose simply detach the hook-and-loop tabs (crocodile) and cut the straps to the required length. Then re-attach the hook-and-loop tabs and thread the straps through the clip closures (Fig. 2).

2. Open the arm sling by detaching the hook-and-loop tabs at the top (Fig. 3). The patient may also open the sling later without undoing the whole immobilizer, in order to readjust the arm's position in the open sling.
3. If necessary, loosen the third strap in the elbow area of the sling.
4. Close all three straps of the arm sling and ensure that a secure, stable but also comfortable fit of the lower arm is achieved without being painful (Fig. 4).
5. Ensure a correct predetermined ante-/retro-positioning of the shoulder by moving the cushion, if necessary.
6. Close the shoulder strap (with shoulder padding) over the healthy shoulder at the front. Correct the length, if necessary, and pull the strap tightly or loosen it a little at the clip closure. Pull the waist strap around the waist and make the adjustments as described in Point 5 (Fig. 1).

Note: The PSA-2 should sit firmly, however it should not be uncomfortable. It is best to check the fit when lying down in order to prevent undesired retroversion.

CARE:

Pillow cover and straps can be washed up to 40° C. Close all hook-and-loop closures completely. Use fine/wool cycle, no fabric softener, wool detergent. Wash separately. Air dry. The foam lining cannot be washed.

FOR SINGLE PATIENT USE ONLY.

~~LATEX~~ NOT MADE WITH NATURAL RUBBER LATEX.

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COMPOSITION:

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

~~LATEX~~ NOT MADE WITH NATURAL RUBBER LATEX.

COMPOSITION:

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

~~LATEX~~ NOT MADE WITH NATURAL RUBBER LATEX.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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