## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**INTENDED USER PROFILE:** The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

**INTENDED USE/INDICATIONS:** To aid in immobilization and protected range of motion associated with ACL, PCL, LCL, and MCL surgeries and meniscal repairs.

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**CONTRAINDICATIONS:** Unstable knees requiring operative therapy, open wounds, cellulitis or lacerations that could be opened by the brace. Do not use if you are allergic to any of the materials contained within this product.

**WARNINGS AND CAUTIONS:** If you experience any pain, swelling, sensations changes, or any unusual reactions while using this product, consult your medical professional immediately. Loss of circulation, patient discomfort and patient re-injury are potential effects caused by device failure. ROM settings are potential effects caused by device failure. Do not use over open wounds, cellulitis or lacerations that could be opened by the brace. Do not use if you are allergic to any of the materials contained within this product.

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**NOTICE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

**APPLICATION INFORMATION:**

- Unfasten strap ends near black buckles and open brace flat by pulling both uprights apart.
- Place leg on top of flat brace. Shorter upright is applied to the thigh. Adjust brace until each upright is aligned to centerline of the medial and lateral side of the leg and the center of the hinge is aligned with the center of the knee joint.

3. Telescopis sliders by depressing the slider button until the desired length is reached. The top and bottom sliders should be adjusted first, followed by the sliders closest to the knee (note that inner sliders cannot be adjusted until the outer sliders are moved to provide sufficient clearance). Ensure that both bars are equal in length and that each strap slider is indexed to the same position on each upright respectively.

4. Adjust the strap lengths:  
- Ensure the strap is lifted from the strap, allowing the strap to move freely (see arrow position in figure 4)  
- By pulling strap away from the body, remove slack from the backside of brace (see arrow position B in figure 4)

- Ensure uprights are aligned on medial and lateral sides of leg  
- Lower tab and apply pressure to secure strap

5. Attach each of the black buckles. Tighten straps by pulling on loose end. Remove excess strap and repeat on other side.

6. Adjust the hinge until the Flexion and Extension stops outward rotation until the desired angle aligns with the button center. Release stops and ensure they are fully inserted.

7. To remove brace, release each buckle. Brace can then be re-applied as a single unit.

**OTHER ADJUSTMENTS:** The hinge bars may be bent to add varus or valgus contouring. Bend each bar by holding thigh/calf bar firmly against a solid surface and apply gentle and constant pressure to the hinge in the direction desired. Bend each bar an equal amount above and below the hinge. For optimal ease of application post-operatively, pre-fit the brace prior to surgery if possible.

**STRAP REPLACEMENT:** Should straps require replacement, carefully use scissors to cut away the strap where it connects to the slider. Insert the new strap and secure. Contact DJO for the X-ROM Replacement Straps Kit.

**CLEANING INSTRUCTIONS:** Hand wash foam liners in water (30°C) with mild detergent. Rinse thoroughly. AIR DRY only, not hot dry.

Regular cleaning of the brace is recommended.  
Composition: Nylon 50%, Aluminum 40%, Polypropylene 5%, Delrin 3%, Stainless Steel 2%.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**RISK ONLY.**  
**INTENDED FOR SINGLE PATIENT USE.**

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**NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY AND DURABILITY, THE OVERALL COMFORT THIS DEVICE IS ONLY ELEMENT OF THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL; THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.**

**CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE AS A CLASS II MEDICAL DEVICE. IT IS THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.**

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