ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS: The Procare MaxTrax Walker is designed to provide support and immobilization to the lower leg, ankle and foot. It may be used for the treatment of stress fractures of the lower limb, stable ankle and/or foot fractures, acute lateral ankle sprains and Achilles tendon repair. Providing immobilization or controlled movement of the limb or bony segment.

CONTRAINdications: This device is contraindicated for unstable fractures of the lower leg, ankle and foot.

PRESCRIBING INFORMATION: Note: Due to the molded outer sole, the orthopedic shoe must be slightly higher than a normal street shoe; therefore an athletic shoe or a street shoe with a 1/2" heel, may be worn if necessary for patient comfort. A heel lift also may be used in the shoe worn on the non-injured foot.

INFORMATION & PRESCRIPTIONS: Note: Due to the molded outer sole, the orthopedic shoe must be slightly higher than a normal street shoe; therefore an athletic shoe or a street shoe with a 1/2" heel, may be worn if necessary for patient comfort. A heel lift also may be used in the shoe worn on the non-injured foot.

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NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to use of this device.

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APPLICATION INFORMATION: For first time application, loosen all straps 1. Untie the strap under the liner.

2. Untie the strap under the liner pads are provided for patient comfort.

3. Place the liner in the boot. At cutouts, fold side panels of liner and separate. Smaller pad should be placed in liner first. At cutouts, fold side panels of liner and secure it either side of liner. For a snug fit around ankle area when applying correctly. Large pad may be applied directly to liner or layered on top of small pad as needed. Application is same as small pad. (Fig. 1)

3. Place foot in liner and secure with contact closure. Make sure the heel goes snugly into the posterior portion of the liner. Secure the footrest with the anterior portion of the liner. Fija la lengüeta delantera al pie, al final del pie.

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4. Both uprights may be shortened by 2" to accommodate various patient heights. To Breakdown Uprights: Secure uprights on edge of table.

5. Remove the straps from the liner.

6. The two straps at the ankle joint may be adapted to accommodate different leg lengths. (Fig. 5)

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