1

2

3

4

5

6

7

8

9

10

1

2

3

4

5

6

7

8

9

10

RX ONLY

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GARANTIE

**ENGLISH**  
BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY.  
CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

**INTENDED USER PROFILE**  
The intended user should be a licensed medical professional, the patient or the caregiver of the patient.  
The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the instructions for use.

**INTENDED USE/INDICATIONS**  
This brace is intended for wrist treatment:  
To provide support and protection for major injury to the wrist.  
To assist the man and the protection in case of lesions graves of the poignet.

**Indications requiring immobilization, support and stabilization of the wrist and distal forearm:**

- Wrist fractures
- Posttraumatic period
- Conservative, pre-operative and postoperative care
- Contraindications
- Providing stabilization to the limb or body segment.

**CONTRAINDICATIONS**

- Any known allergies to the medical devices materials. Patients with known allergies to foams, adhesives, or plastic products.

**WARNING & PRECAUTIONS**

- This product must be fitted by a healthcare professional. The frequency and duration of use should be determined by your healthcare professional.
- Do not heat the brace while it is on the body.
- Do not use the brace if it is not too hot prior to application.
- Brace should be appropriately used to the patient to ensure proper function.
- Care should be taken with patients with delicate or compromised skin, especially the elderly, not to apply the brace too tightly, which may cause pressure and shear to fragile tissue. PATIENTS SHOULD BE INSTRUCTED ON APPROPRIATE TENSIONING DURING ONGOING USE. An under sleeve of stockings may be used for protection.

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