ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

PLAYMAKER EVEREST™ / PLAYMAKER EVEREST™ PRO

- Playmaker Everest™ : Hinged knee brace
- Playmaker Everest™ Pro : Knee Brace with adjustable hinges

INTENDED USER PROFILE

- The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to:
- Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

- Playmaker Everest™ - Indications requiring support and stabilization of the knee joint, such as: mild to moderate lateral ligament instabilities, meniscus lesions, patellofemoral pain, osteoarthritis, general knee pain or swelling, post-op use following knee surgery.

- Playmaker Everest™ Pro - Indications requiring support, stabilization and range-of-motion control of the knee joint, such as: mild to moderate ligament instabilities, meniscus lesions, patellofemoral pain, osteoarthritis, general knee pain or swelling, post-op use following knee surgery. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS

- Do not use if you are allergic to any of the materials contained in this product.

WARNINGS AND PRECAUTIONS

- We recommend that the first application be made under supervision of a health care professional.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.

Note: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS

- 1) If using cold therapy option, pull the undersleeve over the knee (Fig. 1a). Place the Knee Cuff around the knee, with the patella opening centred on the knee cap. The valve at the top of the Knee Cuff (Fig. 1b).
- 2) Place the hinges into the pockets provided (Fig. 2a). Position on each side of the Knee Cuff, using the hook and loop (Fig. 2b). Align the centre of the hinges with the centre of the knee cap. Ensure the 2 hinges are level on each side of the Knee Cuff. If needed, the hinges can be formed to the patient's needs.
- 3) Attach the extension straps to the Thigh & Calf straps on the Knee Cuff. Wrap the extension straps over the Knee Cuff and hinges to ensure a secure fit.

Use WITHOUT the hinges : During the treatment course, and upon advice of a health care professional, the Knee Cuff can also be used without the hinges and extension straps. Secure the Knee Cuff by attaching the thigh Strap and the calf Strap using the hook and loop closures.

4) Adjustable Hinges

- To adjust flexion/extension, open the cap of each hinge, position the flexion/extension stops at the desired angle, and close the hinge cap (Fig. 4a).
- To avoid damaging the hinges, the flexion/extension stops must be positioned at the same angle on both hinges (Fig. 4b).

CARE - KNEE CUFF

- Wrap may be hand washed and air dried.
- Gel inserts may be cleaned with water, moist towel or sponge.

COMPOSITION

KNEE CUFF

Inflatable Pad: Polyester, nylon
Pump: Silicone, Polyethylene, PVC, ABS Resin (thermoplastic polymer), synthetic rubber and steel wire

COLD PACK

Gel: propylene glycol, sodium carboxymethyl cellulose, water.
Fabric: cotton, nylon
Hinges: Aluminium, polyethylene, nylon
Undersleeve: Polyester

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